Bravo Portable II

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GB Instructions for use

- 1.) Affix standard hand piece and angle onto the motor, without jutting out intra spray tubes or light wiring.
- 2.) Connect motor cable with outlet socket (**pict. 1**) to the accumulator. The device is now ready for use.
- 3.) The device is switched on by turning the speed control (**pict. 2**). As soon as the device is switched on, the control lamp flashes.
- 4.) The speed of rotation of the **Bravo Portable II** is infinitely variable between **1,600 - 25,000 rpm**.
- 5.) The direction of rotation can be changed from counter-clockwise (switch **R**) to clockwise (switch **F**).
- 6.) The accumulator gets charged by connecting the battery charger to a 230 V plug and to the charging plug underneath the accumulator. The control lamp flashes. Charging time of the accumulator is approx. 6 hours. The device has to be switched off when charging. In order to secure a long lifetime, the accumulator should be discharged properly (however, not too deeply) before charging.
- 7.) **Warranty**
The producer ensures that the product is free of defects and that it shows all guaranteed features and/or all characteristics agreed upon. The warranty starts with the day of transfer of risk or delivery of the product and is valid for twelve months. Excluded from the warranty are faults and/or defects caused by operational and usual wear, handling error, careless use of the product, connection to unsuitable power supply, operation with the wrong voltage, acts of God, such as fire, lightning, humidity, etc. In case of justified complaints the producer has the right to repair the product, to deliver spare parts or to decrease the sales price. Cancellation of the sales contract can only be claimed by the customer after two repairs have been carried out without success.

Attention!

- Bravo Portable II is for **extraoral** use only.
- Do not lubricate the motor
- Do not drop accumulator, hand piece or angle
- **Bravo Portable II** must not run without hand or angle piece
- Rotating direction may be changed only when the motor is not running
- Do not use newly lubricated hand or angle pieces. Let the newly lubricated hand or angle pieces drain vertically on an absorbent surface for at least 30 minutes before use.

TECHNICAL DATA:

Power supply:	NI-NH storage battery	Rotation:	infinitely variable from 1,600 - 25,000 rpm
Battery voltage:	15 V	Battery lifetime:	> 500 charges
Power input		Charging time:	6 hours / operation time approx. 8 hours
of micromotor	max. 700 mA	Battery voltage of charging device:	230 V

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