Airsonic® Mini-Sandblaster

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Mini Sandblaster only for Extra-Oral Use

Instruction Manual

The Airsonic Mini-Sandblaster significantly enhances the adherence of cements by enlarging the size of the accordingly treated surface.

Installation: The sandblaster requires compressed air of 4-8 bar. In case a dry air compressor is not available, a condensed water separator has to be installed. For installation, you need a T-fitting and a connecting hose. (The technical service of your dental dealer will assist you, if necessary.)

1. Cut existing compressed air line, install T-fitting
2. Connect the female coupling with external thread (optional REF 901 200) to the T-fitting.

- Turbine connection Kavo, Sirona-Titan, Rotoquick (W&H), Bien Air Unifix and NSK are available optionally. The adapter is simply affixed to the existing turbine coupling and connected to the Airsonic with quick coupling.
- Couplings for laboratory connections and EMX Box (old design) are available separately.

Operation: The device is ready for use after connection to the compressed air line according to instruction manual (see above). For sandblasting of the various (extraoral) surfaces, use Airsonic Alu-Oxyd Powder with 50 micron (REF 605 084) or 90 micron (REF 605 086). The powder should be in absolutely dry, fine-grained condition. Fill the transparent container at the hand piece ¾ full. Important: When connecting the turbine coupling, the foot controller has to be operated, too. The hand piece is held like a ball-pen and the push button is actuated with the finger. The distance between nozzle and object should be approx. 2-10 mm. The sandblasting process is activated by light pressing of the valve head. The stronger the pressure on the button, the stronger the abrasive pressure of the sandjet. The hand piece nozzle can be turned by 360° in order to adjust the optimal angle between nozzle and object. In case the sandblaster is clogged by moist powder, remove the powder container, cover the nozzle and activate the push button repeatedly in short intervals.

Warranty Period: The manufacturer guarantees that the contractual product is flawless and features all warranted and/or agreed characteristics. The warranty period starts from the date of transfer of perils or with delivery of the product for a twelve months' period. Excluded from warranty are faults and/or damages, which are e.g. due to operational wear, normal wear, improper use, operating errors, careless product handling by the customer, connection to inappropriate power sources, operation with wrong kind of current, force majeure like fire, lightning, damage due to damp, etc. In case of justified defect claims, the manufacturer has the right of repair, replacement or reduction of purchase price. The customer can only claim cancellation of the sales contract after two ineffective repairs by the manufacturer.

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HÄGER & WERKEN GmbH & Co. KG