GLASPARELHEATER

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GLASS BEAD HEATER

GB Instructions

Purpose: The glass bead heater is designed to heat instruments prior to their use. It can also be used to keep mouth mirrors mist-free.

ATTENTION: The device may not be used for the general sterilisation/disinfection of instruments during or as a substitute for the preparation process!

WARNING: Ensure that only prepared/sterilised instruments are placed inside the glass bead heater. If instruments that are obviously contaminated (e.g., during or following use on a patient) are placed inside the glass bead heater, then the glass bead heater must be sterilised and the glass beads must be replaced immediately after the treatment.

The device features particularly good insulation characteristics resulting in minimum electricity consumption. After approximately 25-30 minutes it reaches a constant temperature (230° C) which is maintained by a special thermostat.

Use:

1. The container is filled with glass beads.
2. Connect the device to the power supply (230 V).
3. Flip on the power switch.
4. The red indicator lamp illuminates and only goes off once the temperature of 230° C has been attained. The green indicator lamp illuminates to indicate that the device is ready for use. Since the device is equipped with a precision thermostat, the red indicator lamp blinks on and off at regular intervals to indicate that the device is working.
5. Heat the instruments in the glass beads for 10-15 seconds prior to their use and allow them to cool to body temperature.
6. The device can remain in operation for the entire day since the temperature is automatically regulated.

Important!

1. The housing for the device becomes warm when it is heating, but not hot.
2. Avoid all contact with the internal container, as it also heats up to the device's working temperature of 230° C.
3. Prior to their use, ensure that the instruments have not been damaged or limited in their performance capability as a result of the heating process. We therefore recommend that you only use stainless steel instruments in combination with the glass bead heater.
4. For hygiene reasons, we recommend replacing the glass beads approximately every 6 months. To replace the glass beads, you must first disconnect the device from the power supply and wait until the glass beads inside have cooled.

Glass Bead Refill Pack - Mini REF 452 301
Glass Bead Refill Pack - Maxi REF 452 304

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