**Side-effects:**

With proper use of this medical device, un-wanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

**Contra-indications/interactions:**

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.

**Storage:**

Store in a cool dry low humidity area away from direct sunlight.

Packages: Syringe 2 ml

**Attention:**

The product is designed for onetime use. When reusing the product the functionality cannot be guaranteed and a high risk of infection occurs.

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### Instructions for use

#### Description:

After Bleaching Care is a water-based cream for remineralization and desensitization after the bleaching process. Pleasant peppermint flavor.

#### Directions for use:

Professional application: immediately following surgery bleaching.

#### I. Mirafluor tray application:

1. Extrude a generous layer of After Bleaching Care into the tray and apply to the upper and / or lower teeth.
2. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
3. Remove the tray.
4. Ask the patient to rinse any After Bleaching Care remaining on the surface. Any residual After Bleaching care can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

#### II. Non tray application:

1. If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet.
2. Apply a generous layer of After Bleaching Care to the tooth surfaces using an application swab, gloved finger or difficult interproximal areas using an Tooth Cleaning Brush.
3. Leave After Bleaching Care undisturbed in the mouth for a minimum of 3 minutes.
4. Ask the patient to rinse any After Bleaching Care remaining on the surface. Any residual After Bleaching Care can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

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GDF GmbH
Vertrieb: Häger & Werken GmbH & Co. KG