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EN

**Super-Splint (De Luxe) is a multiple layer fiberglass strip for strengthening dental resins.**

**Intended use:**

- Splinting loose anterior teeth caused by periodontitis or trauma.
- Re-implantation of extracted or totally dislocated anterior teeth to act as a long-term temporary bridge.
- Reinforcing direct or laboratory-made crowns and bridges.
- Reinforcing acrylic denture repairs.

**INSTRUCTIONS FOR USE:**

1. Place a rubber dam.
2. Clean all involved teeth with non-fluoride polishing paste.
3. Etch the oral and interdental surfaces of the teeth for 60 seconds then rinse and blow dry.
4. If necessary, the teeth can be fixed temporarily by placing a small drop of bonding agent into each of the interdental spaces.
5. Coat the oral and interdental surface with a bonding agent.
6. Determine the required length of the Super-Splint (De Luxe) strip with a periodontal probe and cut it to size.
7. Impregnate it with the bonding agent on glass slab.
8. Place Super-Splint (De Luxe) strip onto the oral surface and hold it in place with the application clips.
9. The strip is kept in position while polymerized 30 – 40 seconds per interproximal space.
10. Any rough areas can be smoothed with a layer of flowable composite. The splint is then trimmed and polished. Remove the rubber dam and check the occlusion.

**Notice: The product is for single use only. In case of a re-use its functionality can no longer be guaranteed and will highly increase the danger of infection. Medical device.**

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© 2019 HAGER & WERKEN