D

Mirawhite® Gingiva Protector

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Mirawhite® Gingiva Protector

Instructions for use

Gingiva Protector is a light-curing gingiva guard which is used similar to a rubber dam for isolating the gingiva from the teeth requiring treatment.

Indications: For covering the gingiva during bleaching in the dental practice.

Contraindications: Intolerance to the ingredients.

Preparation: Take appropriate measures (e.g. use cheek and lip retractors) to ensure that tongue and lips are kept in a safe position during treatment (and the subsequent bleaching procedure) and do not get into contact with the material. Remove the seal of the syringe and replace it with the syringe application tip. Control the flow of the material.

Application: Dry the gingiva. Apply Gingiva Protector to a width of approx. 4 – 6 mm and a thickness of 2 mm along the area of application. Make sure to press the material into the interdental spaces. Cure for 20 to 40 sec. using a standard light-curing hand lamp. Do not hold the lamp too long onto one area. After curing, remove any excess material onto the teeth using a scalpel. The bleaching gel can now be applied.

Finishing: After bleaching, remove the Gingiva Protector by lifting the material at one end with an instrument, such as a probe, and pull it off. Remove the syringe application tip and replace it with the seal.

Shelf life/storage: Gingiva Protector should not be used after the expiry date. Store Gingiva Protector in a dark place. Avoid heat and direct sunlight.

The products are for single-use only. Their functionality can no longer be guaranteed if used more than once. Furthermore, a re-use leads to a high risk of infection.

This product contains dimethacrylate and must not be used in cases of known allergies or intolerances to this type of material.

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