# FIT-N-SWIPE CLEAN

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## GB Adhesive cleaning pad for clean instruments

- Microfibre with high retention capacity
- High cleaning effect and absorbency
- Practical and can be used with one hand
- Can be positioned individually in the immediate working area

**Intended use: Self-adhesive disposable cleaning pads for cleaning instruments and mouth mirrors during dental treatment.**

**Indication: The use of the pads is indicated for treatments in the following areas: scaling, preparation, examination, fillings, cementing, surgery, root canal treatment, periodontal therapy, microsurgery.**

 **Please note!** Pads are NOT STERILE and must be STERILISED in a certified autoclave according to the instructions for use for textile fabrics BEFORE USE.

Method	Process	Temperature	Minimum hold time	Drying time
Superheated steam	Vacuum process (fractionated pre-vacuum)	132 °C	4 min.	20 min.
		135 °C	3 min.	

Pointed and sharp objects can puncture/cut the pad.

Pad adhesion is proven on vinyl, nitrile, latex. Special glove coatings may reduce the adhesiveness.

## Instructions for use:

- Bend the pad to facilitate removal of the silicone paper.
- Press the pad firmly onto the desired surface. All surfaces (glove, rubber dam) must be absolutely dry. Do not stick the pad directly on the skin and do not reuse it.
- A new pad must be used if the position is changed or a new patient is involved.
- Any residue of disinfectant or soap must be removed.
- For cleaning, carefully wipe the mouth mirrors and instruments on the cleaning pad. Do not press sharp and pointed areas into the pad.

**Disposal:** Used pads must be disposed of as contaminated waste in accordance with official regulations.

All serious incidents related to the product must be reported immediately to the manufacturer and the competent authority of your Member State.

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© BEXPLORA