FIT-N-SWIPE DRY

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GB

Adhesive drying pad for clear mouth mirrors

- Multi-layer fabric
- High cleaning power and absorbency
- Removes even greasy or oily dirt without streaking
- Practical and can be used with one hand
- Can be positioned individually in the immediate working area

Intended use: Self-adhesive disposable drying pads for drying instruments and mouth mirrors during dental treatment.

Indication: The use of the pads is indicated for treatments in the following areas: scaling, preparation, examination, fillings, cementation, surgery, root canal treatment, periodontal therapy, microsurgery.

Please note! Pads are NOT STERILE and must be STERILISED in a certified autoclave according to the instructions for use for textile fabrics BEFORE USE.

Method	Process	Temperature	Minimum hold time	Drying time
Superheated steam	Vacuum process (fractionated pre-vacuum)	132 °C 135 °C	4 min. 3 min.	20 min.

Pointed and sharp objects can puncture/cut the pad.

Pad adhesion is proven on vinyl, nitrile, latex. Special glove coatings may reduce the adhesiveness.

Instructions for use:

- Bend the pad to facilitate removal of the silicone paper.
- Press the pad firmly onto the desired surface. All surfaces (glove, rubber dam) must be completely dry. Do not stick the pad directly on the skin and do not reuse it.
- A new pad must be used if the position is changed or a new patient is involved.
- Any residue of disinfectant or soap must be removed.
- To dry, carefully wipe mouth mirrors and instruments on the drying pad. Do not press sharp and pointed areas into the pad.

Disposal: Used pads must be disposed of as contaminated waste in accordance with official regulations.

All serious incidents related to the product must be reported immediately to the manufacturer and the competent authority of your Member State.

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© BENPLORA
HAGER & WERKEN GmbH & Co. KG