H&W LOGIC
H&W PANORAMA II
H&W OTG II

Schutzbrillen	Gebrauchsanweisung
Protective Glasses	Instruction for Use
Lunettes protectrices	Mode d'emploi
Beschermbril	Gebruiksaanwijzing
Occhiali di protezione	Istruzioni per l'uso
Gafas de protección	Información de uso

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GB Instructions for use

Intended use:

The protective glasses ensure eye protection in grinding, lathe, milling, fine mechanical and laboratory operations as well as easy installation operations and are, therefore, suitable to be used in dental laboratories and dental practices.

To be followed prior to use of PPE:

- **Make sure the product is suitable for your intended use** (Follow these instructions precisely). Examine the eyewear and in particular the lenses, to make sure there is no sign of damage. Pay attention to defects like scratches, fractures, etc. which generally have a negative impact on the functionality such as missing impact resistance or limited vision. If necessary, replace faulty lens, visor or glasses.
- Due to transferring impacts of high-speed particles the protective glasses might put the spectacle wearer to risk if worn over prescription spectacles.
- The safety eyewear must only be used at room temperature for protection against high-speed particles. Must not be used at extreme temperatures.
- **Use PPE according to its intended use and ensure it corresponds to the risks at work.**
- **The PPE is a personal protective eyewear and must not be worn by more than one user.**
- **The PPE has to be replaced should it get damaged during use, whether by hit, fire or corrosion.**
- **Always store contaminated PPE in a separate container, separating it from non-contaminated PPE until getting disposed of or getting disinfected.**
- **Clean the PPE regularly and do not put back into packaging without examining all components for intactness following each use.**
- Clean the PPE with a soft cloth and water only. Disinfectants must not be used for the cleaning

- of the PPE. Please use our Astro Clean (spray disinfectant REF 554 301).
- Store the PSA at temperatures of 5° C and 40° C in a dry and dust-free surrounding in its original packaging. Keep in a dry place and do not expose to direct sunlight or heat in order to keep the produce free from damage.
- Keep the instructions of use and the PPE in its original packaging.
- In conformity with Directive 89/686/EEC and EN 166 standard specifications. Please find the PPE's identification code stated on its packaging.
- Details regarding the PPE's suitability are stated on the product and can also be found on the packaging.
- Please find explanation for the abbreviation in the following manual ("Characteristics of the glasses")
- The complete pair of safety glasses has to be classified with the lower grade if symbols F and B do not refer to both, the lens and the frame.

Any changes to the PPE not carried out by the manufacturer as well as any failure to adhere to the instructions will be equated with improper and not permitted use. Any replacement equipment has to be ordered from Hager & Werken. It is for Hager & Werken to decide whether a part can be substituted or replaced. Defect components, resulting from improper handling, are non-returnable.

Characteristics of the glasses:

An abbreviation according to harmonised standard EN 166 is imprinted on the protective glass: HW 1 F CE. The temple arms are imprinted with the following abbreviation: HW EN166 F CE. You also find the model details given on the temple arms (6-digit row of numbers).

- HW** Abbreviation for the European representative – Responsible within the EU:
Hager & Werken GmbH & Co. KG, Ackerstr.1, 47269 Duisburg, Germany
- 1** Optical class – For work requiring extra good visual function
- F** Mechanical strength – Resistant to high-speed (45m/s) particles
- CE** Conformity Mark – Type-examination certificate is available for the products, in conformity with European Guideline 89/686/EEG (personal protective equipment)
- EN 166** Conformity Mark – Products are in accordance with harmonised standard EN 166

355 616

355 617 Model description – (matches the REF number)

355 618

Durability:

If given instructions regarding use, storage and cleaning are observed, the article can be used for approximately 6 months after removal of packaging. It is recommended to exchange the product after five years' storage; earlier in case of defects. It is not permitted to use the PPE outside the areas of application stated in these instructions of use and on the packaging. Sensitive persons wearing the PPA may suffer from allergic reactions if their skin comes into contact with the material.

Parkson Safety Industrial Corp., 4F-21, Sec 5, No. 5, Sinyi Rd, Taipei City 110, Taiwan

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Helmut Feldtmann GmbH, Zunftstraße 28, 21244 Buchholz
Distribución Hager & Werken GmbH & Co. KG, Ackerstr. 1, 47269 Duisburg