## DE OXYSAFE Liquid Professional – Gebrauchsanweisung

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### Darreichungsform von OXYSAFE Liquid Professional

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10°C

**CE 0044**

Exclusively distributed by: HAGER & WERKEN GmbH & Co. KG

Ackerstraße 1 · 47269 Duisburg, Germany

T +49 (203) 9926 9-0 · F +49 (203) 299283 · [www.hagerwerken.de](http://www.hagerwerken.de)

Ardoz Healthcare BV

Kasteleinenkampweg 9 C

5222 AX 's-Hertogenbosch, The Netherlands

---

## EN OXYSAFE Liquid Professional – Instructions for Use

---

### **What is OXYSAFE Liquid Professional?**

OXYSAFE Liquid Professional is a product with a patented active oxygen technology. This technology becomes active at the moment of contact with the skin or mucous membrane.

### **How does OXYSAFE Liquid Professional work?**

OXYSAFE Liquid Professional contains the patented Ardox-X<sup>®</sup> technology which becomes active at the moment of contact with the skin or mucous membrane. This technology is based on a stabilised hydrocarbon-oxoborate complex in which a non radical species of oxygen is released under the influence of saliva. The hydrocarbon-oxoborate complex functions as a matrix that releases active oxygen on the area to be treated. The oxygen-enriched microenvironment has a proven positive effect on the (infected) wound treatment, e.g. infected gingival pockets.

### **For which purpose can you use the OXYSAFE Liquid Professional?**

OXYSAFE Liquid Professional is used in the treatment of gingivitis, periodontitis and peri-implantitis.

### **How do you use the OXYSAFE Liquid Professional?**

OXYSAFE Liquid Professional is being used in treatment of gingivitis, periodontitis and peri-implantitis.

### **When must OXYSAFE Liquid Professional not be used?**

With the exception of titanium, OXYSAFE Liquid Professional must not be used if metal implants or prosthesis are present. Make sure to remove any prosthesis prior to using OXYSAFE Liquid Professional.

### **How do you use OXYSAFE Liquid Professional?**

#### Dosage

Rinse and/or brush your mouth three times a day for one minute with 10 ml of OXYSAFE Liquid Professional and spit out. Do not rinse with water afterwards.

#### Prevent contamination

To prevent contamination, always use the provided measuring cup and rinse it with water afterwards.

### **How long can OXYSAFE Liquid Professional be used?**

Discontinue the treatment once you have recovered from the periodontal condition and note that its use should not exceed seven years.

### **What to consider during pregnancy.**

Do not use during pregnancy/breastfeeding.

### **Additional information on OXYSAFE Liquid Professional**

The critical ingredient in OXYSAFE Liquid Professional is the Ardox-X<sup>®</sup> technology; based on an oxygen-releasing complex.

Contains Sodium Perborate. This product is a Medical Device.

### **Dosage form of OXYSAFE Liquid Professional?**

OXYSAFE Liquid Professional is a clear to slightly opaque colourless liquid.

The bottle contains 250 ml OXYSAFE Liquid Professional.

**Any serious incident that occurs in relation to the use of the product must be reported to the manufacturer and to the competent authority.**

**This user information was last revised 04-20.**

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### **Comment agit OXYSAFE Liquid Professional ?**

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#### **Comment utiliser OXYSAFE Liquid Professional ?**

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