Hager Outback

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EN Instructions for use

Intended use:

The protective glasses “Hager Outback“ ensure eye protection in grinding, lathe, milling, fine mechanical and laboratory operations as well as easy installation operations and are, therefore, suitable to be used in dental laboratories and dental practices.

Additionally they offer a 100% eye protection against UV-light (EN170-2002).

Neck strap insertion:

Use hot water to warm both, the tip of the plastic part on the neck strap and the temple tips, for about 10 – 20 seconds, and then press both parts into each other.

Nose bridge adjustment:

The glasses need to fit securely: The protective glass should fit to the face as closely as possible. The glasses are delivered with a narrow nose bridge. It can be slightly widened by pushing it apart.

To be followed prior to use of PPE:

- **Make sure the product is suitable for your intended use** (Follow these instructions precisely). Examine the eyewear and in particular the lenses, to make sure there is no sign of damage. Pay attention to defects like scratches, fractures, etc. which generally have a negative impact on the functionality such as missing impact resistance or limited vision. If necessary, replace faulty lens, visor or glasses.
- Due to transferring impacts of high-speed particles the protective glasses might put the spectacle wearer to risk if worn over prescription spectacles.
- The safety eyewear must only be used at room temperature for protection against high-speed particles. Must not be used at extreme temperatures.
- **If the PPE is delivered disassembled, the user has to assemble the components according to separate instructions, ensuring the characteristics of the glasses match the stated standards. Accessories have to be mounted regularly and carefully. The PPE has to fit securely.**
- **Use PPE according to its intended use and ensure it corresponds to the risks at work.**
- **The PPE is a personal protective eyewear and must not be worn by more than one user.**
- **The PPE has to be replaced should it get damaged during use, whether by hit, fire or corrosion.**
- **Always store contaminated PPE in a separate container, separating it from non-contaminated PPE until getting disposed of or getting disinfected.**
- **Clean the PPE regularly and do not put back into packaging without examining all components for intactness following each use.**
- Clean the PPE with a soft cloth and water only. Disinfectants must not be used for the cleaning of the PPE. We recommend for the disinfection our Astro Clean (REF 554 301).
- Store the PSA at temperatures of 5° C and 40° C in a dry and dust-free surrounding in its original packaging. Keep in a dry place and do not expose to direct sunlight or heat in order to keep the produce free from damage.
- Keep the instructions of use and the PPE in its original packaging.
- The conformity of the directive (EU) 2016/425 with the requirements of the EN166:2001 is warranted. Please find the PPE's identification code stated on its packaging.
- Details regarding the PPE's suitability are stated on the product and can also be found on the packaging.
- **Please find explanation for the abbreviation in the following manual (“Characteristics of the glasses“)**

Strictly follow the instructions of use when assembling the PPE's components. Any changes to the PPE not carried out by the manufacturer as well as any failure to adhere to the instructions will be equated with improper and not permitted use. Any replacement equipment has to be ordered from Hager & Werken. It is for Hager & Werken to decide whether a part can be substituted or replaced. Defect components, resulting from improper handling, are non-returnable.

Häger Outback

EN Instruction for use

Characteristics of the glasses:

An abbreviation according to harmonised standard EN166:2001 and EN170:2002 is imprinted on the protective glass: 2C-1.2 HW 1 F CE. The temple arms are imprinted with the following abbreviation: HW EN166 F CE. You also find the model details given on the temple arms.

2	UV-Filter: Glasses have a level 2 UV protective filter in accordance with EN170:2002.
C	View: Good colour recognition
1.2	Protection level: Glasses have a 1.2 ultraviolet protective filter in accordance with EN170:2002
HW	Abbreviation for the European representative: Responsible within the EU: Häger & Werken GmbH & Co. KG, Ackerstr.1, 47269 Duisburg, Germany
1	Optical class: For work requiring extra good visual function
F	Mechanical strength: Resistant to high-speed (45m/s) particles
CE	Conformity Mark: Type-examination certificate is available for the products. The conformity with European directive (EU)2016/425 and policy 2001/95/EG (personal protective equipment) is granted.
EN166:2001	Conformity Mark: Products are in accordance with harmonised standard EN166:2001
Mod.	Model description: Last 3 digits of REF number

Durability:

If given instructions regarding use, storage and cleaning are observed, the article can be used for approximately 6 months after removal of packaging. It is recommended to exchange the product after three years' storage; earlier in case of defects. It is not permitted to use the PPE outside the areas of application stated in these instructions of use and on the packaging. The PPE includes material which might lead to allergic reactions in persons being highly allergy-prone.

Warning notice:

By replacement or assembly of multiple single parts to a complete PPE, the protection level meets at best the protection of the single component with the lower protection class.

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Hager Werken GmbH & Co. KG