HÄGER
WERKEN

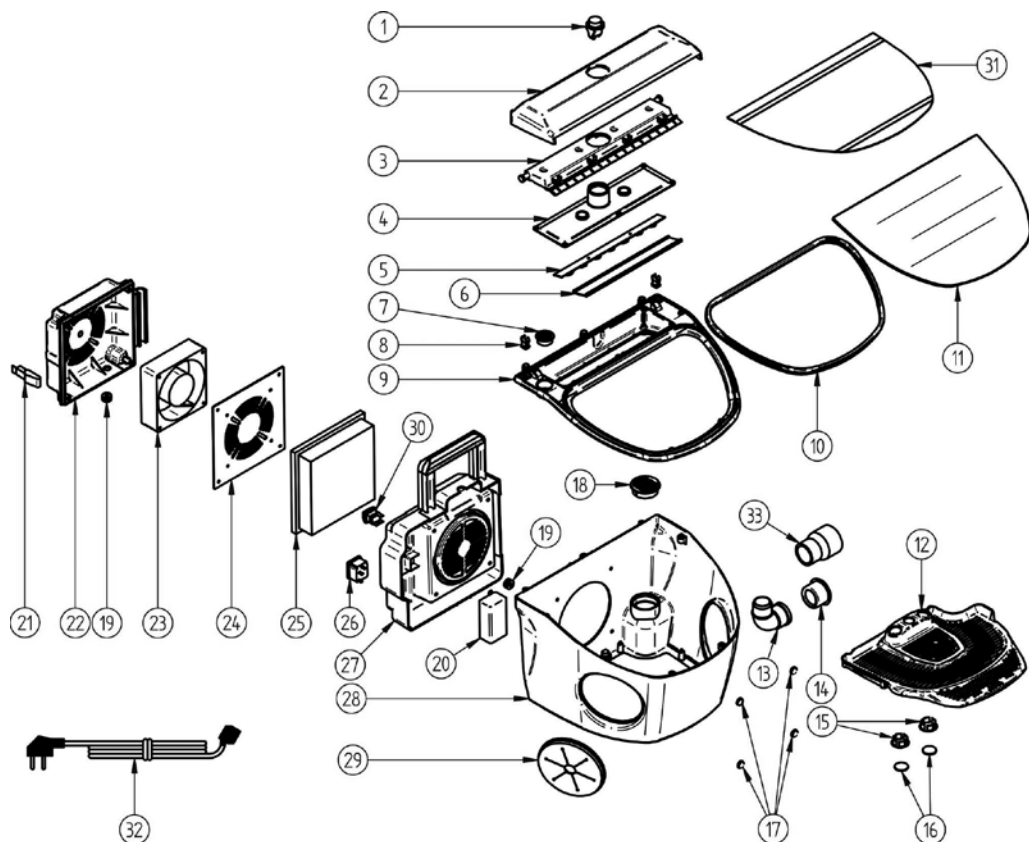
AIRSONIC® ABSORBO BOX® II

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Instructions for Use

Mode d'emploi

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Zeichnung Nr. Drawing number Dibujo no.	Hager & Wer- ken REF	Ersatzteile und Zubehör	Replacement and ascesory parts	Pièces de rechange et pièces accessoires	Piezas de recambio / adicionales
11	401106	Glasdeckel	Glass lid	Couvercle en verre	Tapa de cristal
12	401109	Ablage-Tray	Tray	Plateau	Bandeja
25	401092	Filter	Filter	Filtre	Filtro
29	401107	Gummimanschette, ein- zeln	Single rubber sleeve	Manchon en caoutchouc simple	Manguito de goma simple
31	401108	Schutzfolie für Glas- scheibe, 10 St.	Protective foil for glass lid, 10 pcs.	Feuille de protection pour le couvercle en verre, 10 pièces	Plástico protector cristal, 10 u.
32	401111	Netzkabel 230 V	Cable 230 C	Câble 230 C	Cable 230 V

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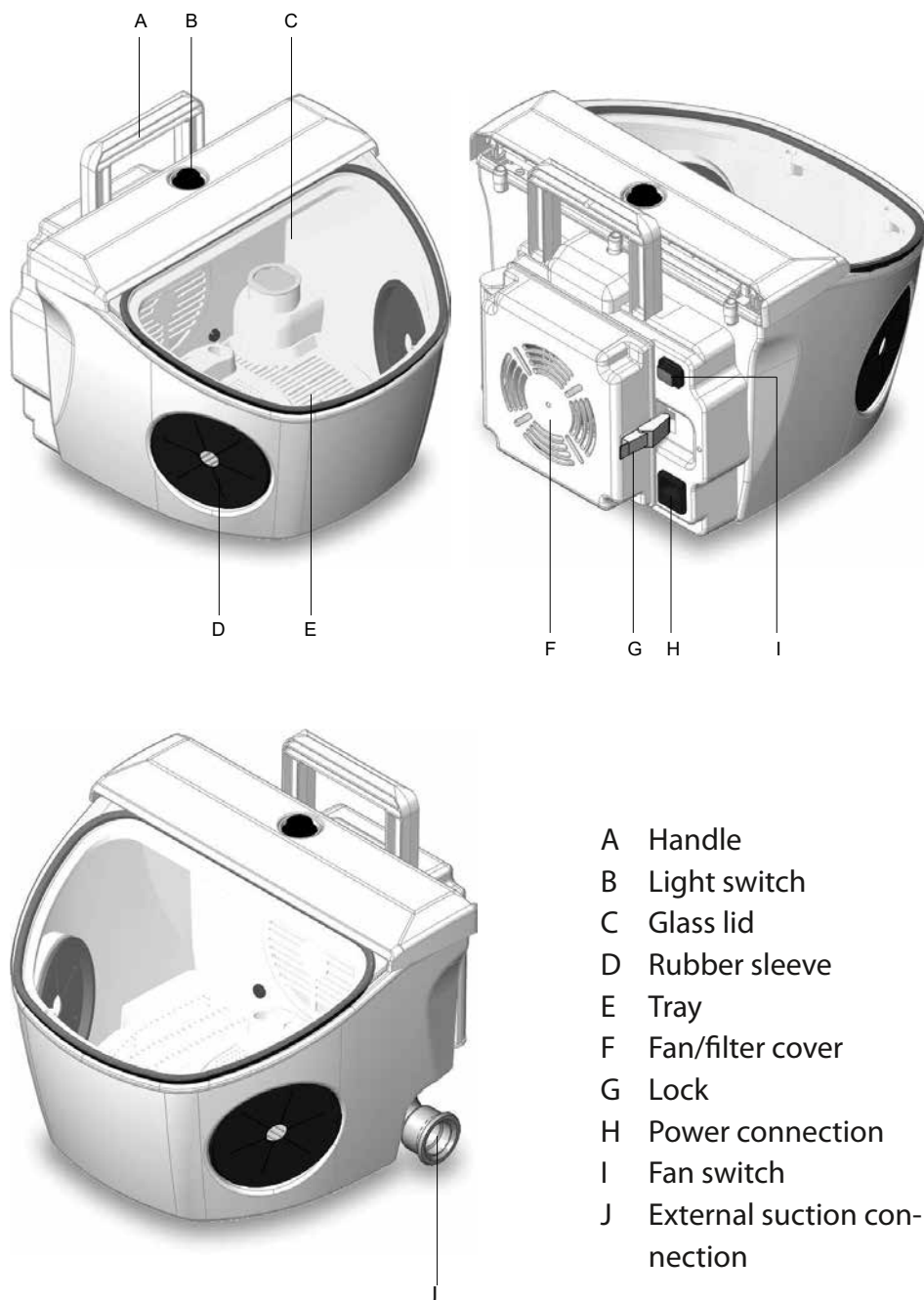
EN Instruction for Use

Blasting cabinet with suction Airsonic® Absorbo Box® II

EN Instruction for Use

The AIRSONIC ABSORBO BOX II finishing is specially designed for the reworking of small parts in the dental prosthesis sector. To achieve optimal performance and product performance, we recommend that you carefully read the following instructions:

Installation

- Carefully unpack the unit.
- Choose a rigid, flat surface, away from sources of vibration or heat, for the location of the unit. Place the unit at a height that allows you to comfortably handle objects inside the chamber.
- Connect the electrical plug to a 230 V, 50/60 Hz power supply with an earth connection.
- The cabin has its own fan and particle filter, but can also be used connected to an external aspiration. To do this, remove the plug that is placed on the outlet tube at the rear (J). The suction you use must be capable of causing the smooth and continuous circulation of the dust particles generated towards the outlet tube.
- Use a protective plastic glued to the inside of the glass.

REF 401108 (10 pieces)

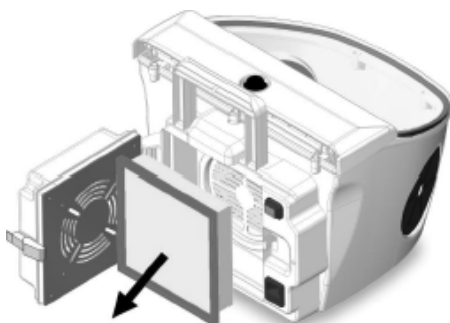
Maintenance and cleaning

To guarantee a long life of the unit, we recommend:

- Use a protective plastic glued to the inside of the glass.
- REF 401108 (10 units.)
- Periodically remove dust from inside the cabin by lifting the interior tray (E).
- To carry out general cleaning of the device, simply use a soft cloth.

Cleaning and replacing the filter

1. Open the latch (G) to access the filter.
2. Pull the filter out for cleaning or replacement (Ref. 401092).

Operation

To work in good conditions, we recommend:

1. Open the cab lid and insert the part you want to blast inside. It is also possible to insert the pieces through the side openings.
2. Always work with the glass lid closed.
3. Protect yourself properly: wear protective goggles, gloves and a mask.
4. Press the switch (H) to start the forced suction.
5. Press the switch (B) to turn on the light inside the chamber.

Precautions

- Keep this manual in an appropriate place for future reference.
- Check the general condition of the device immediately after removing the packaging.
- Failures caused by poor installation or misuse of the device are excluded from the warranty.
- Always work with the cabin lid closed, and with appropriate protections.
- Always work with forced aspiration.
- Protect yourself adequately, use a mask, gloves and safety glasses.
- To avoid the risk of electric shock, this equipment should only be connected to a suitable grounded power source. Before connecting the appliance, check that the identification label indicates the correct voltage. Installation must be in accordance with all applicable safety standards.
- Remove any dirt or water from the cable connection.
- Avoid pulling, twisting or excessively bending the network connection cable.
- Do not touch the appliance or the cable with wet hands.
- Install the product away from heat sources, dust, high humidity, water splashes and flammable materials (alcohol, solvents, etc.).
- Unplug the appliance immediately if you detect noise, odor or smoke.
- Prevent children or non-specialized personnel from manipulating the device.
- If in doubt, do not operate the device. Contact a qualified and authorized repairer.
- Do not disassemble, repair or modify the appliance beyond cleaning and maintaining filters and fuses.
- Unplug the appliance in the event of a storm and when not in use for a long time.
- Unplug the appliance before cleaning it, do not wet it directly, do not use chemicals.
- Contact your dealer for questions or detailed troubleshooting information.
- Before using the device, make sure that it is sitting on a firm and solid base.
- The cable can only be replaced by one with the same characteristics as that supplied by the manufacturer.
- Do not use the machine outside its intended use.

Technical characteristics

Height:	300 mm
Width:	360 mm
Length:	385 mm
Weight:	4.38 kg
Power supply:	230 V, 50/60 Hz
Consumption:	22 W

Declaration of conformity UE

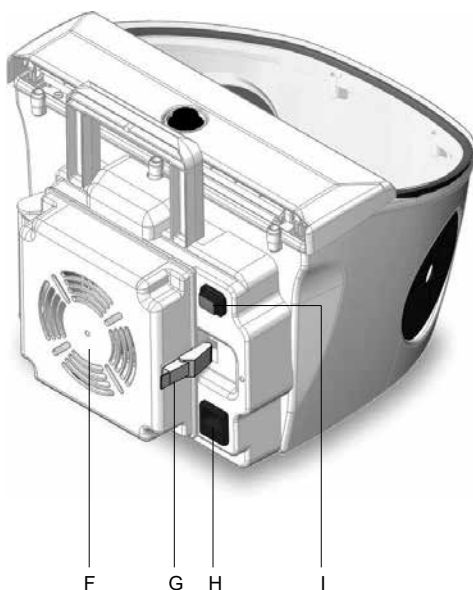
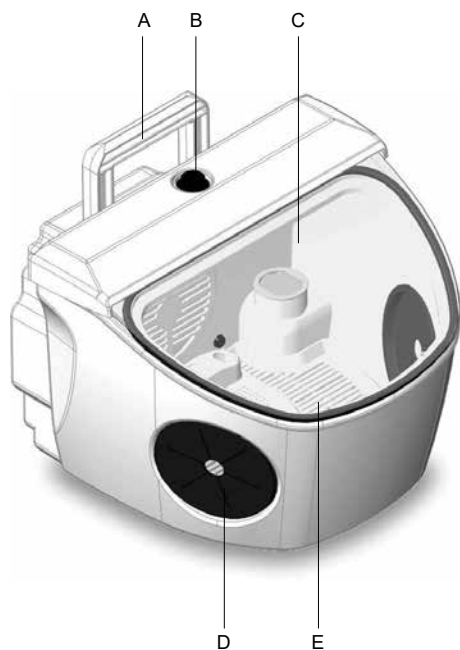
The firm Hager & Werken GmbH & Co KG, Ackerstr.1, 47269 Duisburg, Germany hereby declares that the product AIRSONIC ABSORBO BOX II, REF 401105 meets the requirements set out in the following directives:

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HÄGER & WERKEN GmbH & Co. KG