Reso-Pac® 20 x 2 g**REF 155024**

DE	2
EN	3
FR	4
NL	5
ES	6
IT	7
PT	8
EL	9
BG	10
CZ	11
HR	12
HU	13
SK	14
SL	15
RO	16
PL	17
SV	18
DK	19
NO	20

HÄGER & WERKEN GmbH & Co. KG
 Ackerstraße 1
 47269 Duisburg, Germany
 T +49 (203) 99269-0
info@hagerwerken.de
www.hagerwerken.de

CH REP

Tecor-Dental AG
 Ebnatstrasse 65
 8200 Schaffhausen
 Schweiz

DE

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INSTRUCTIONS FOR USE:

INTENDED USE

Odontological wound dressing as a mechanical barrier for wounds and sutures.

INDICATION:

Reso-Pac is suitable for covering wounds after tooth extractions and surgical procedures if the wound needs to be protected from mechanical influences and immobilisation of the adjacent tissue is not guaranteed. Reso-Pac can also be used to cover applied medication so that it can also be used, for example, for wound care as part of anticoagulant treatment. Due to its composition, the product adheres very well to moist and even bleeding surfaces.

CONTRAINDICATIONS:

During pregnancy and breastfeeding, the products may be used only on medical advice. Undesirable side effects in connection with the use of the product have not been observed to date. Interactions with other preparations are possible, an individual risk assessment by a doctor is necessary. The product must not be used in the case of known hypersensitivity or allergies to any of the ingredients.

INTENDED PATIENT GROUP:

The product is suitable for use on humans. This includes women and men, adolescent and adult patients and children aged 12 and over.

COMPOSITION:

Sodium carboxymethyl cellulose, polyvinyl acetate, ethyl alcohol, petrolatum, polyethylene oxide resin

INSTRUCTIONS FOR USE:

- 1.) Remove Reso-Pac from the cup.
- 2.) Form Reso-Pac into a thin roll (running water may make this easier) and stick it to the area to be treated.
- 3.) A medication can also be applied under the wound dressing, which can then act on this area for hours.
- 4.) The surface of the applied wound dressing should be smoothed with a well-moistened finger.

STORAGE:

Store in a dry place and protect from sunlight. Maintain storage temperatures of 0 – 25 °C.

IMPORTANT:

The product is intended for use by dental professionals. Without chewing or manipulation, Reso-Pac sticks for approx. 2 hours and should then be removed by the patient independently. Advise patients that they must not eat or drink after applying Reso-Pac while the paste is in their mouth. Do not use in children under 12 years of age. No risks are to be expected if the product is swallowed in typical application quantities. The product is intended for single use. If the product is reused, its functionality is not guaranteed and there is a risk of infection. All serious incidents occurring in connection with the product must be reported immediately to the manufacturer and the competent authority of your Member State. Store in a dry place. Protect from sunlight. Maintain storage temperatures of 0 – 25 °C. The product and its contents should be disposed of in accordance with official, national and regional regulations.

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