mirafluor®-gel Minze **REF** 605800
mirafluor®-gel Erdbeere **REF** 605801

DE	2
EN	3
FR.....	4
NL.....	5
ES.....	6
CS	7

HÄGER & WERKEN GmbH & Co. KG
Ackerstraße 1
47269 Duisburg, Germany
T +49 (203) 99269-0
info@hagerwerken.de
www.hagerwerken.de

CH **REP**
Tecor-Dental AG
Ebnatstrasse 65
8200 Schaffhausen
Schweiz

MD **CE 0197**    25°C

DE

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Fluoridgel mit 12300 ppm Fluorid

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Fluoride gel with 12300 ppm fluoride

Intended use:

- For caries prophylaxis with sodium fluoride.
- For remineralisation of the tooth enamel.

Indication:

- Risk of caries / Initial caries.
- Demineralisation of tooth enamel.

Ingredients (mirafluor®-gel mint):

Water, sorbitol, sodium fluoride, disodium hydrogen phosphate, hydrogenated castor oil (PEG-40), hydroxyethyl cellulose, phosphoric acid, flavour.

Ingredients (mirafluor®-gel strawberry):

Water, sorbitol, sodium fluoride, disodium hydrogen phosphate, hydrogenated castor oil (PEG-40), hydroxyethyl cellulose, phosphoric acid, sodium saccharin, flavour.

Instructions:

Gently brush the gel onto the tooth surface (hazelnut-sized amount) or apply using the Mirafluor tray in an amount appropriate for the respective application spoon. Ensure sufficient contact time between the gel and teeth (2 – 4 minutes). Then rinse the mouth briefly.

Contraindication:

This product should not be used in persons who do not have control over the swallowing reflex. Use is contraindicated if there are wounds or exfoliation of the oral mucosa. These products may be used in pregnant and breastfeeding women only on medical advice. Undesirable side effects associated with the use of the product have not been observed to date. Interactions with other products are possible; an individual risk assessment by a doctor is necessary. If used too frequently or with contact time that is too long or in the event of additional fluoride intake, there is a risk of (dental) fluorosis. Do not use this product in cases of known hypersensitivity or allergies to one of the ingredients.

Caution:

This product is intended for use by dental professionals. Avoid swallowing the gel. Do not use more often than twice a month.

Consult a doctor in case of overdosage or swallowing the gel in larger amounts. All serious incidents occurring in connection with the product should be reported immediately to the manufacturer and the competent authority of your Member State. The product is intended for use in humans. Do not use in children under 6 years of age. Store in a dry place at room temperature. Protect from sunlight. Do not refrigerate, do not freeze. Larger quantities of the product and its contents should be disposed of in accordance with official, national and regional regulations. Use within 3 months after opening.

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