Miratray® Implant, Intro-Kit	REF	101250
Miratray® Implant, OK S1	REF	101251
Miratray® Implant, OK S2	REF	101252
Miratray® Implant, OK S3	REF	101253
Miratray® Implant, UK I1	REF	101254
Miratray® Implant, UK I2	REF	101255
Miratray® Implant, UK I3	REF	101256

DE	2
EN	4
FR.....	6
NL.....	8
ES.....	10
PT.....	12
IT	14
SV.....	16
PL.....	18
CS	20
DA	22
SL.....	24
HR	26
SK.....	28
BG	30
EL.....	32
HU	34
NO	36
RO	38
RU	40

HÄGER & WERKEN GmbH & Co. KG
 Ackerstraße 1
 47269 Duisburg, Germany
 T +49 (203) 99269-0
info@hagerwerken.de
www.hagerwerken.de

CH REP

Tecor-Dental AG
 Ebnatstrasse 65
 8200 Schaffhausen
 Schweiz

UK RP

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EN

INSTRUCTIONS FOR USE

IMPLANT IMPRESSION TRAY WITH FOIL TECHNIQUE

Intended use

Miratray Implant is a universal, customised, single-use implant impression tray and holder for dental impression materials, designed to facilitate taking impressions of structures within the human oral cavity for the purpose of documenting and/or replicating anatomical conditions.

Indication

Miratray Implant is used in place of a custom implant tray and can be used for both edentulous and dentate jaws in the following indications:

- Impressions for producing implant-supported crowns, bridges and dentures
- Transfer of the exact implant axis and depth for preparing models
- Functional impressions for implant-supported full or partial dentures
- Impressions for multiple implants in a single jaw requiring high precision (e.g. screw-retained constructions)

Contraindications

The product is contraindicated in patients with allergies or intolerances to one or more of its components.

Basic working principle

The special transparent film on the base of the tray makes it easier to select the correct tray size and ensure a proper fit. Miratray Implant impression trays do not need to be individually adjusted around the implant abutments, as the film is simply pierced during the impression-taking process. The film then fits snugly against the abutments, preventing any unwanted leakage of impression material. This ensures a quick, clean and precise implant impression.

Selection of the Implant Tray

The adequate tray size can be selected either by conventional intraoral measurement with a measuring probe or by simple visual fitting. Sufficient space (min. 2 – 3 mm) for the impression material should be left between the impression tray and the abutments respectively the alveolar process. For dentulous jaws, the locking screws and pins have to be chosen in a way that they sit sufficiently above the occlusion surfaces of the adjacent teeth, otherwise a piercing of the foil is not or only partially possible.

Preparation of the Impression Tray for Impression Taking

You should always use a tray adhesive. For a better adjustment of the tray filled with impression material in the mouth, the exact points of abutment penetration can be marked with a pen when selecting the adequate Miratray Implant tray.

Impression Material

A silicone or polyether impression material with high final hardness is recommended for impression. The instructions of the respective manufacturer have to be followed when working with the impression material.

Disinfection of Impression

The impression has to be disinfected after removal from the patient mouth.

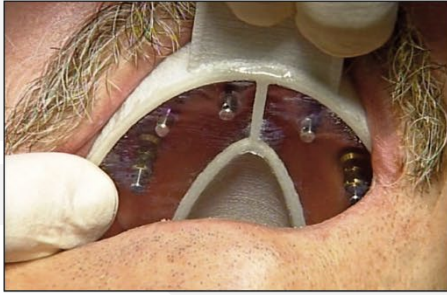
Hazard Warning

Use the tray with undamaged foil only. Cracks in the foil allow impression material to leak, which may result in incorrect impressions.

Commercial Packing

Introduction kit with three impression trays each (S, M, L) for maxilla and mandible (dentulous and edentulous). Refill packs (6 pcs.) for the respective size.


The product is for single use only. In case of a re-use its functionality can no longer be guaranteed and will highly increase the danger of infection. The manufacturer and the competent authority of your member state must be immediately notified of all serious incidents occurring in connection with the product.

Situation for impression. Fitting of the transparent Miratray Implant impression tray. The position of the implants can be marked.

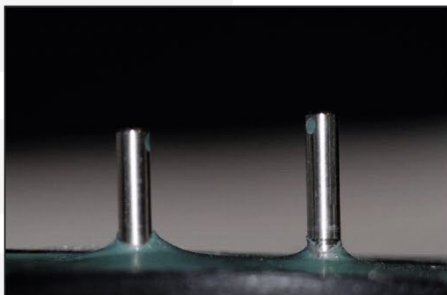Piercing of the foil with a light, steady pressure.

The clean point of penetration of the locking pins.

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