Miraject Endo Luer (0,5 x 40mm)	25 Gauge Ø: 0,5 mm, Länge 40 mm	REF 254215
Miraject Endo Luer (0,3 x 40mm)	30 Gauge Ø: 0,3 mm, Länge 40 mm	REF 254222
Miraject Endotec Duo (0,6 x 25mm)	23 Gauge Ø: 0,6 mm Länge: 25 mm	REF 254230
Miraject Endotec Duo (0,5 x 25mm)	25 Gauge Ø: 0,5 mm Länge: 25 mm	REF 254231
Miraject Endotec Duo (0,4 x 25mm)	27 Gauge Ø: 0,4 mm Länge: 25 mm	REF 254232
Miraject Endotec Duo (0,3 x 25mm)	30 Gauge Ø: 0,3 mm Länge: 25 mm	REF 254233
Miraject Endotec Luer (0,8 x 25mm)	21 Gauge Ø: 0,8 mm Länge: 25 mm	REF 254216
Miraject Endotec Luer (0,6 x 25mm)	23 Gauge Ø: 0,6 mm Länge: 25 mm	REF 254217
Miraject Endotec Luer (0,5 x 25mm)	25 Gauge Ø: 0,5 mm Länge: 25 mm	REF 254218
Miraject Endotec Luer (0,3 x 25mm)	30 Gauge Ø: 0,3 mm Länge: 25 mm	REF 254219
Miraject Endotec Luer (0,4 x 25mm)	27 Gauge Ø: 0,4 mm Länge: 25 mm	REF 254228
Miraject PL Super	20 Gauge Ø: 0,9 mm Länge: 22 mm	REF 254214

DE	2
EN	4
FR.....	6
NL.....	8
ES.....	10
PT.....	12
IT	14
SV.....	16
PL.....	18
CS	20
DA	22
SL.....	24
HR	26
EL.....	28
ET.....	30
HU	32
NO	34
RU	36

HÄGER & WERKEN GmbH & Co. KG
 Ackerstraße 1
 47269 Duisburg, Germany
 T +49 (203) 99269-0
info@hagerwerken.de
www.hagerwerken.de

CH REP
 Tecor-Dental AG
 Ebnatstrasse 65
 8200 Schaffhausen
 Schweiz

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EN

INSTRUCTIONS FOR USE

DISPOSABLE DENTAL APPLICATION CANNULAS WITH LUER CONNECTORS

Intended use:

Sterile, disposable application cannulas with a Luer connector for the flushing and/or administration of liquids, gels and pastes in dental applications. The product is intended for single use only and must be used exclusively by qualified dental professionals.

Indication:

REF	Product name	Variant (dimensions)	Indication
254215	Miraject Endo Luer (0.5 x 40 mm)	25 Gauge Ø: 0.5 mm, length 40 mm	Application of liquids and pastes into the root canal
254222	Miraject Endo Luer (0.3 x 40 mm)	30 Gauge Ø: 0.3 mm, length 40 mm	Application of liquids and pastes into the root canal
254230	Miraject Endotec Duo (0.6 x 25 mm)	23 Gauge Ø: 0.6 mm, length: 25 mm	Flushing of root canals and periodontal pockets via two lateral openings
254231	Miraject Endotec Duo (0.5 x 25 mm)	25 Gauge Ø: 0.5 mm, length: 25 mm	Flushing of root canals and periodontal pockets via two lateral openings
254232	Miraject Endotec Duo (0.4 x 25 mm)	27 Gauge Ø: 0.4 mm, length: 25 mm	Flushing of root canals and periodontal pockets via two lateral openings
254233	Miraject Endotec Duo (0.3 x 25 mm)	30 Gauge Ø: 0.3 mm, length: 25 mm	Flushing of root canals and periodontal pockets via two lateral openings
254216	Miraject Endotec Luer (0.8 x 25 mm)	21 Gauge Ø: 0.8 mm, length: 25 mm	Flushing of root canals and periodontal pockets via a lateral opening
254217	Miraject Endotec Luer (0.6 x 25 mm)	23 Gauge Ø: 0.6 mm, length: 25 mm	Flushing of root canals and periodontal pockets via a lateral opening
254218	Miraject Endotec Luer (0.5 x 25 mm)	25 Gauge Ø: 0.5 mm, length: 25 mm	Flushing of root canals and periodontal pockets via a lateral opening
254219	Miraject Endotec Luer (0.3 x 25 mm)	30 Gauge Ø: 0.3 mm, length: 25 mm	Flushing of root canals and periodontal pockets via a lateral opening
254228	Miraject Endotec Luer (0.4 x 25 mm)	27 Gauge Ø: 0.4 mm, length: 25 mm	Flushing of root canals and periodontal pockets via a lateral opening
254214	Miraject PL Super	20 Gauge Ø: 0.9 mm, length: 22 mm	Application of liquids and gels, and for flushing of particularly large root canals

Contraindication:

The product must not be used if you have a known hypersensitivity or allergy to any of its ingredients.

Instructions for use:

Use only in combination with suitable Luer syringes. Before use, follow the instructions for use of the products to be combined.

Don suitable protective gloves before use. Open the packaging and remove the product. Do not remove the protective cap from the cannula until just before use. Secure the cannula firmly to the syringe by placing it on the syringe, pressing down gently and twisting it into place. Before use, ensure that the product is securely connected and firmly in place. Then remove the front protective cap immediately before use. Carry out the treatment or flushing in accordance with the recommended procedures. During the procedure, ensure that there is no loosening, detachment or leakage. Dispose of used cannulas immediately after use in a suitable, puncture-resistant container for sharp and pointed medical waste. Disposal must be carried out in accordance with the applicable national, regional and local regulations governing medical waste.

 Warning:

This product is intended for use by qualified dental professionals. Do not use if the packaging or the sterile barrier has been damaged, opened, contaminated or tampered with, or if the product is damaged. Use immediately after opening the sterile packaging and removing the protective cap. Do not bend the cannula. Do not touch the cannula tube directly. Do not allow the cannula tube to remain in contact with the administration or flushing fluids for any prolonged period. Prolonged exposure may lead to crystallisation, which could block or damage the cannula tube. Do not place the protective cap back onto the cannula by hand after use. Use a suitable device to ensure the protective cap is securely replaced. The cannula is intended for single use only. Do not reuse or re-sterilise it, as this may lead to malfunction, infection or cross-infection.

Any serious incidents occurring in connection with the product must be reported promptly to the manufacturer and the relevant authority in your Member State. The product is intended for use on humans. Store in a dry place. Protect from direct sunlight and high temperatures.

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