Miraject Luer 17/42	Gauge: 25 Ø: 0,5 mm Länge: 42 mm	REF 254210
Miraject Luer 17/23	Gauge: 25 Ø: 0,5 mm Länge: 23 mm	REF 254211
Miraject Luer 30/42	Gauge: 27 Ø: 0,4 mm Länge: 42 mm	REF 254212
Miraject Luer 30/23	Gauge: 27 Ø: 0,4 mm Länge: 23 mm	REF 254213

DE	2
EN	3
FR	4
NL	5
ES	6
PT	7
IT	8
PL	9
RU	10

HÄGER & WERKEN GmbH & Co. KG
Ackerstraße 1
47269 Duisburg, Germany
T +49 (203) 99269-0
info@hagerwerken.de
www.hagerwerken.de

CH REP
Tecor-Dental AG
Ebnatstrasse 65
8200 Schaffhausen
Schweiz

DE

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EN

INSTRUCTIONS FOR USE

DISPOSABLE DENTAL INJECTION CANNULAS WITH LUER CONNECTORS

Intended use:

Sterile, disposable injection cannulas with a Luer connector for the injection of fluids in dental applications. The product is intended for single use only and must only be used by qualified dental professionals.

Indication:

The disposable injection needle is indicated for the administration of dental injections, in particular for the injection of local anaesthetics as part of dental treatments requiring local anaesthesia.

Contraindication:

The product must not be used if you have a known hypersensitivity or allergy to any of its ingredients.

Instructions for use:

Use only in combination with suitable Luer syringes. Before use, follow the instructions for use of the products to be combined.

Don suitable protective gloves before use. Open the packaging and remove the product. Do not remove the protective cap from the cannula until just before use. Secure the cannula firmly to the syringe by placing it on the syringe, pressing down gently and twisting it into place. Then remove the front protective cap immediately before use. Administer the injection in accordance with the recommended procedures. During use, ensure that there is no loosening, detachment or leakage. Dispose of used cannulas immediately after use in a suitable, puncture-resistant container for sharp and pointed medical waste. Disposal must be carried out in accordance with the applicable national, regional and local regulations governing medical waste.

 Warning:

This product is intended for use by qualified dental professionals. Do not use if the packaging or the sterile barrier has been damaged, opened, contaminated or tampered with, or if the product is damaged. Use immediately after opening the sterile packaging and removing the protective cap. Do not bend the cannula. Do not touch the cannula tube directly. Do not allow the cannula tube to remain in contact with the administration or flushing fluids for any prolonged period. Prolonged exposure may lead to crystallisation, which could block or damage the cannula tube. Do not place the protective cap back onto the cannula by hand after use. Use a suitable device to ensure the protective cap is securely replaced. When replacing the protective cap, take care to ensure that the tip of the cannula does not come into contact with the inside of the protective cap. The cannula is intended for single use only. Do not reuse or re-sterilise it, as this may lead to malfunction, infection or cross-infection.

Any serious incidents occurring in connection with the product must be reported promptly to the manufacturer and the relevant authority in your Member State. The product is intended for use on humans. Store in a dry place. Protect from direct sunlight and high temperatures.

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