Miraject P
Miraject P Super

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EN

INSTRUCTIONS FOR USE

DISPOSABLE DENTAL APPLICATION CANNULAS FOR CYLINDRICAL AMPOULES

Intended use:

The dental cartridge cannula is designed for use with suitable dental cartridge syringes and dental cylinder ampoules, and is intended for the application of paste-like materials in dental procedures. The product is for single use only and must be used exclusively by qualified dental professionals.

Indication:

The dental cartridge cannula is indicated for the precise application of paste-like dental materials in endodontic, periodontal and restorative treatments.

Contraindication:

The product must not be used if you have a known hypersensitivity or allergy to any of its ingredients.

Instructions for use:

Use only in combination with suitable dental cartridge syringes and dental cylinder ampoules. Before use, follow the instructions for use of the products to be combined.

Don suitable protective gloves before use. Break the tamper-evident seal with a quick sideways movement. Remove the rear protective cap and screw the cannula onto the dental cartridge syringe with the cartridge end facing outwards. When connecting the cannula to the dental syringe, do not apply excessive torque to the protective cap. Before use, ensure that the product is securely connected and firmly in place. Then remove the front protective cap immediately before use. Administer the product in accordance with the recommended application procedure. During treatment, ensure that there is no loosening, detachment or leakage. Dispose of used cannulas immediately after use in a suitable, puncture-resistant container for sharp and pointed medical waste. Disposal must be carried out in accordance with the applicable national, regional and local regulations governing medical waste.

 Warning:

This product is intended for use by dental professionals. Do not use if the packaging, the sterile barrier or the tamper-evident seal has been damaged, opened, contaminated or tampered with, or if the product is damaged. Use immediately after opening the sterile packaging and breaking the tamper-evident seal. Do not bend the cannula. Do not touch the cannula tube directly. Do not allow the cannula tube to remain in contact with the administration media for an extended period. Do not replace the protective cap on the cannula by hand after use. Use a suitable device to ensure the protective cap is securely replaced. The cannula is intended for single use only. Do not reuse or re-sterilise it, as this may lead to malfunction, infection or cross-infection.

Any serious incidents occurring in connection with the product must be reported promptly to the manufacturer and the relevant authority in your Member State. The product is intended for use on humans. Store in a dry place. Protect from direct sunlight and high temperatures.

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